



## Document Control

Title	<b>Series and Image Number Settings</b>		
Doc Number	T10007-9002		
Product	All CR Control Stations		
Revision	1.0		
Date	07/18/2005		
KC number	N/A	KC Ref. number:	N/A

<input checked="" type="checkbox"/> Final	<input type="checkbox"/> Tentative	<input checked="" type="checkbox"/> Internal Release	<input checked="" type="checkbox"/> External Release
---	------------------------------------	--	--

## Applicable Equipment Information

Equipment Model	Serial Numbers	Firmware Version	Software Version
CS-1, CS-2, CS-3	ALL	N/A	ALL

Submitted by Jeff Anders

KMMI Approved by Jeff Anders

<input type="checkbox"/> Change of parts	<input type="checkbox"/> Return parts	<input checked="" type="checkbox"/> Final action	<input type="checkbox"/> Tentative action
--	---------------------------------------	--	---

### 1. Overview:

This document will describe the behavior differences that occur when changes are made to the Manage Serial Property, Image Separator, and Apply Level inputs in the Function Setup <Maker> menu of the 09610 service tool.

### ☒ Corrective Action

<input type="checkbox"/>	High Priority & Mandatory
<input type="checkbox"/>	Mandatory
<input type="checkbox"/>	High Priority
<input type="checkbox"/>	Customer Complaint Units
<input type="checkbox"/>	Upon Failure or Problem
<input type="checkbox"/>	Optional



### ☐ Technical Information

<input type="checkbox"/>	Urgent
<input type="checkbox"/>	High Importance
<input checked="" type="checkbox"/>	Normal
<input type="checkbox"/>	
<input type="checkbox"/>	





## 2. Synopsis:

This document provides a table showing the settings needed to send an image to PACS as part of a series or have the image be sent to PACS as its own series.

## 3. Requirements:

N/A

## 4. Related Documents:

N/A

## 5. Work Time Required:

15 Mins



## 6. Procedure

The table below describes the behavior differences that occur when changes are made to the Manage Serial Property, Image Separator, and Apply Level inputs in the Function Setup <Maker> menu of the 09610 service tool.

Manage Serial Property	Image Separator	Apply Level	Results
REGIUS150 Compliance	Image No.	Current Order	All Images in One Series
REGIUS150 Compliance	Image No.	Current Study	All Images in One Series
REGIUS150 Compliance	Image No.	Current Serial	All Images in One Series
REGIUS150 Compliance	Series No.	Current Order	Each Image is its own Series
REGIUS150 Compliance	Series No.	Current Study	Each Image is its own Series
REGIUS150 Compliance	Series No.	Current Serial	Each Image is its own Series
DICOM Compliance	Image No.	Current Order	Series increments with body part change. Images increment within the series they are related to
DICOM Compliance	Image No.	Current Study	Each Image is its own Series
DICOM Compliance	Image No.	Current Serial	Series increments with body part change. Images increment within the series they are related to
DICOM Compliance	Series No.	Current Order	Series increments with body part change. Images increment within the series they are related to
DICOM Compliance	Series No.	Current Study	Each Image is its own Series
DICOM Compliance	Series No.	Current Serial	Series increments with body part change. Images increment within the series they are related to

For most case REGIUS150 Compliance, Image No., and Current Order will be just fine, in the case of Hanging Protocol to PACS we advise using REGIUS150 Compliance, Series No., Current Order. This will send each Image over to PACS as its own series and allow the PACS to use Protocol Name (0018,1030) to hang properly.

The information contained in this document is not to be distributed or copied without prior written approval from KMMI. The information contained in this document is intended for only trained and certified KMMI Personnel and Service Agents. All information is subject to change without notice. The use of this document and its contents are restricted. Its use and distribution is regulated by Konica Minolta Medical Imaging USA, Inc