
WelchAllyn®

12400

StrabismoScope™

Unidirectional Occluder
Cache-œil unidirectionnel
Unidirektioneller okkluder
Ocluser unidireccional
Occlusionatore unidirezionale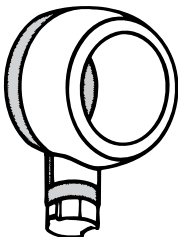

U.S. Patent No. 4,903,706
N° de brevet américain 4 903 706
U.S.-Patent Nr. 4.903.706
Patente estadounidense No. 4.903.706
Brevetto USA n. 4.903.706

OPERATING AND LAMP REPLACEMENT INSTRUCTIONS

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OPERATING INSTRUCTIONS

TO ATTACH TO POWER SOURCE

Align cutouts in the StrabismoScope base with lugs on power source, push in slightly and turn 90°. The StrabismoScope is now ready to use. To remove, reverse this procedure.

Press down on red button of handle while turning the black dial to the end to increase light intensity.

CLEANING AND LAMP REPLACEMENT

CLEANING RECOMMENDATIONS


All external surfaces of the StrabismoScope can be cleaned by using a damp alcohol wipe. To clean the glass, use either an alcohol wipe, or a glass cleaner such as Windex. Do not use abrasive materials on the glass surfaces.

CAUTION:


The StrabismoScope cannot be sterilized by gas or steam autoclave, or by soaking in disinfecting solutions. To insure proper operation, all service (with the exception of lamp replacement and cleaning) should be performed by Welch Allyn.

LAMP REPLACEMENT

Be sure lamp is cool before removing. If the instrument has been in use, it should sit for five minutes before changing the lamp.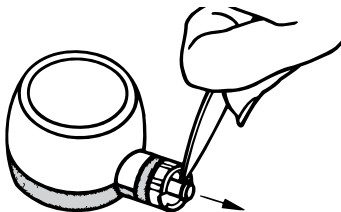

To change the lamp, remove the StrabismoScope from its power source. Remove the lamp from the base of the instrument by prying out with fingernail, nail file, or other instrument.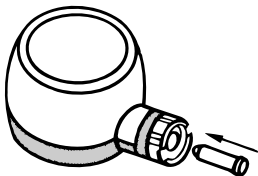

Taking care not to touch the glass surface, replace lamp by inserting lamp into base of instrument so that the lamp “key” drops into either slot. (Push lamp in until firmly sealed.)

Replacement lamp is a Welch Allyn No. 03000 Halogen Lamp.

For Guaranteed Performance
Use Only Genuine Welch Allyn Lamps.

ORDERING INFORMATION

No. 12400 — 3.5v Halogen StrabismoScope

No. 03000 — 3.5v Halogen replacement
lamp for No. 12400

No. 12451 — 3.5v Halogen StrabismoScope
Set* includes: 12400 — 3.5v Halogen
StrabismoScope, 71000 — 3.5v rechargeable
handle, 05231 — Soft carrying case

*Available for **export**. Please specify country and voltage.

This device was invented by A. Aguirre
Vila-Coro, M.D. Madrid, Spain and
A. Aguirre Vila-Coro, M.D. University of
Texas at Houston.

For service, send unit to:

Welch Allyn, Inc.

Technical Service

4341 State Street Road

Skaneateles Falls, New York USA


13153-0220

Instructions for use of the StrabismoScope Unidirectional Occluder


Before You Start

- Dim the room lights slightly.
- Direct the patient's attention to an accommodative target (small toy or wall chart).
- The patient side of the instrument (white) should be positioned close to the patient's eye.




- I. Turn the StrabismoScope on and place it over the **Right Eye**. Watch both the uncovered and covered eyes.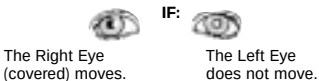

**A Phoria Exists.
Proceed to Section II.**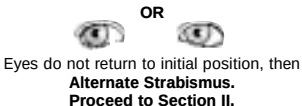

- II.** Turn the StrabismoScope on and place it over the **Left Eye**. Watch both the uncovered and covered eyes.

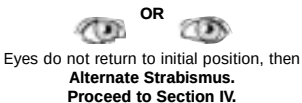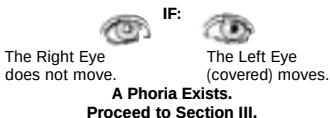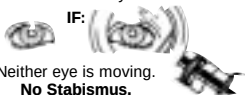IF a phoria is diagnosed, referral to a specialist is indicated if the phoria is large or vertical and the patient is experiencing headaches or vision problems. Many patients have small phorias and these may be no cause for concern, if other symptoms are not present.

IF a tropia is diagnosed, the patient should be referred to a specialist for a thorough evaluation.

III. Confirming Phoria If Diagnosed (or Suspected):

1. Cover one eye with the StrabismoScope turned on. Watch for the covered eye to move.
2. Remove the StrabismoScope. Watch for the eye to return to its initial position. Repeat over the other eye.

IV. Confirming Tropia If Diagnosed (or Suspected):

1. Cover the dominant eye with the StrabismoScope turned on. Both eyes should move.
2. Remove the StrabismoScope.
 - a. Eyes will return to original position (if dominant eye was covered)
 - b. Eyes will remain in the same position (if alternate strabismus is present)

HELPFUL HINTS

1. Be aware of small or “flick” movements of the eye. Small amounts of strabismus are sometimes difficult to see, but these too, often lead to amblyopia. Movements can be in either a horizontal or vertical direction.
2. Children’s attention spans tend to be brief. Vary the toys and targets that are used.
3. If possible, perform this test with the patient focusing at both near and distant targets.

This easy to understand guide is available as a laminated wall poster and is free with your purchase of the StrabismoScope. To receive the poster, return the enclosed coupon.

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13153-0220 U.S.A.