C7115 Apex® Hand Control

ENGLISH	1	简体中文	53
FRANÇAIS	5	POLSKI	57
ITALIANO	9	TÜRKÇE	61
DEUTSCH	13	ROMÂNĂ	65
ESPAÑOL	17	العربية	69
ΕΛΛΗΝΙΚΑ	21	РУССКИЙ	73
PORTUGUÊS	25	HRVATSKI	77
PORTUGUÊS DO BRASIL	29	ČEŠTINA	81
NEDERLANDS	33	MAGYAR	85
DANSK	37	LIETUVIŲ K.	89
SVENSKA	41	SLOVENČINA	93
SUOMI	45	SLOVENŠČINA	97
NORSK	49		

C7115 Apex® Hand Control

REF	Description
C7115	Apex® Hand Control

DESCRIPTION

The C7115 Apex Hand Control is a hand-held device that provides the same control functions as the 10k® Irrigation Console and APEX® Universal Irrigation System (except Stand-by and Presets). The remote gives the user the added advantage to control these functions from within the sterile field. It connects to the Remote connector on the front panel of the pump console. The remote is designed to withstand the harsh temperatures and pressures associated with autoclave sterilization.

Carefully read the 10k Irrigation Console instruction manual prior to operating this equipment.

The C7115 Apex Hand Control shall be returned every 12 months for servicing.

INTENDED USE/INDICATIONS FOR USE

The C7115 Remote is designed to be used with various Linvatec Irrigation Console for control of certain console functions from within the sterile field.

CONTRAINDICATIONS

None known.

WARNINGS

1. Do not use equipment in the presence of flammable anesthetics, gases, disinfecting agents, cleaning solutions, or any material susceptible to ignition due to electrical sparking.
2. The Remote is provided non-sterile, reusable and must be cleaned and sterilized before the first use and after every use.
3. Failure to follow the specified service interval could result in reduced instrument performance.
4. The remote control should be cool to the touch prior to use. A hot remote control can cause a system malfunction which may lead to patient injury.

PRECAUTIONS

1. United States Federal law restricts sale of this device to or on the order of a physician.
2. Handle all equipment carefully. If any equipment is dropped or damaged in any way, return it immediately for service.
3. The warranty becomes void and the manufacturer is not liable for direct or resulting damage if:
 - The device or the accessories are improperly used, prepared or maintained;
 - The instructions in the manual are not adhered to;
 - Non-authorized persons perform repairs, adjustments or alterations to the device or accessories.
4. Clean and sterilize equipment according to instructions for use.
5. After each use, thoroughly clean the Remote.
6. Do not handle the Remote by the cord. Do not pull on the cord to remove it from the surgical console. Damage may occur. Use connector only.
7. There are no user-serviceable parts associated with this remote. Return to CONMED for service, if needed.

8. Do not excessively bend or kink the cord, or roll equipment over the cord. Damage may occur. If any damage is noticed, return to CONMED for service.
9. This is a latex-free product.
10. Do not allow any sharp instruments to come in contact with the remote buttons. Damage may occur.

INSTALLATION INSTRUCTION

WARNINGS:

- **Prior to installing the remote, ensure the contact surfaces of the connector are dry. Moisture in the connector can cause system malfunctions.**
- **Prior to using the remote, ensure it is cool to the touch. A hot remote can cause system malfunctions and lead to patient injury.**

Use surgical aseptic technique to connect the remote to the respective irrigation console.

1. Ensure the main power switch on the pump is turned off. Hold the sterilization container containing the remote in the vicinity of the Scrub.
2. Remove the remote from the sterilization container and place it in the sterile field.
3. Pass the remote connector body to the Circulator. The remainder of the remote and cord stays in the sterile field.
4. Line up the alignment marks on the remote connector body with the remote receptacle on the pump, and insert the connector completely. **Push straight in until fully seated, do not twist the connector.**
5. Ensure the remote door is completely closed prior to use. Attach the clip on the remote cord to a convenient surface in the sterile field.

Any operating difficulties should be reported to your local Sales Representative or CONMED Customer Service.

Inspections Recommendations

1. Inspect Device Prior to use.
2. Generally, unmagnified visual inspection under good light conditions is sufficient. All parts of the devices should be checked for visible soil and/or corrosion.
3. Functional checks should be performed where possible.
4. Connectors should be checked for proper mating.
5. Remove or replace damaged components.

CLEANING AND STERILIZATION INSTRUCTIONS

1. Follow universal precautions for protective apparel when handling and cleaning contaminated equipment.
2. Remove all traces of blood and coagulated material, etc., by wiping with a clean, soft, lint-free cloth dampened with a mild pH-balanced detergent.
3. Prior to any cleaning, sterilization or disinfection, the remote door must be opened to a partially opened position.
4. To clean, immerse remote in warm, soapy water and scrub with a soft brush dampened with a mild pH-balanced detergent. Remove all traces of blood and coagulated material.
5. Rinse under running water. Rinse again with distilled water.
6. Do not immerse or rinse the Remote connector. Damage may occur.
7. Dry with a clean, lint-free cloth.
8. Individually wrap Remote.
9. Follow the recommended minimum sterilization exposure times listed below.
10. Use the recommended tray parameters in its instruction manual when sterilizing remote in a system sterilization tray.

11. Steam sterilization is safe and effective and has no contraindications for sterilizing this equipment. Sterilize per the following table.

Method	Cycle	Minimum Temperature	Minimum Exposure	Minimum Dry Time
Steam (wrapped)	Pre-vacuum	270°F (132°C)	4 minutes	8 minutes
Steam (wrapped)	Gravity	250°F (121°C)	30 minutes	8 minutes

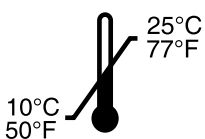
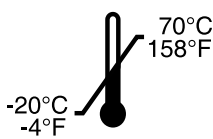
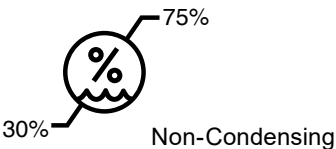
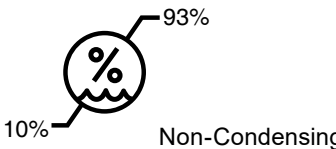
TROUBLESHOOTING

Symptom	Possible Cause	Corrective Action
Remote does not operate	- Cord(s) not connected securely. - Door is not closed. - Remote is faulty. - Remote button stuck down.	- Securely connect Remote to console. - Close door on remote body. - Return for service. - Open remote door and reseat button on door.

TECHNICAL SPECIFICATIONS

Classification:	Class I (intended for connection to Class I surgical console). Type rating of applied part is dependent on surgical console.
Protection Ingress of Fluids:	IPX0 (ordinary)
Mode of Operation:	Continuous
Remote Cable Length:	11 ft. (3.35 m)
Weight:	12.4 oz. (352 g)

ENVIRONMENTAL REQUIREMENTS

Environmental Conditions	Operating	Storage and Transport
Temperature:		
Relative Humidity:		
Atmospheric Pressure:		

PACKAGING and LABELING

Upon initial receipt of the product, this device should only be used if the original packaging and labeling are intact. If packaging has been opened or altered, contact your local CONMED representative, or in the U.S. contact the Customer Service Department.

CONTACT INFORMATION

For more information or a product demonstration, contact your local CONMED sales representative, or call 1-866-426-6633 in the U.S.

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©2022 CONMED Corporation. All rights reserved.
CONMED UK Ltd
CONMED Corporation
MDSS GmbH